

PHILIPS




Quick installation instructions for DreamStation 4G cellular modem

Follow these simple steps to remove your DreamStation 3G cellular modem (if applicable) and install your DreamStation 4G cellular modem. For more information, please check the Instruction for Use manual provided with your DreamStation 4G cellular modem.

A. Instructions for removing your DreamStation 3G cellular modem

Ensure your DreamStation device is turned off.

1. Open the modem accessory door located on the left side.
2. Gently, pull the 3G cellular modem out of the accessory slot.*

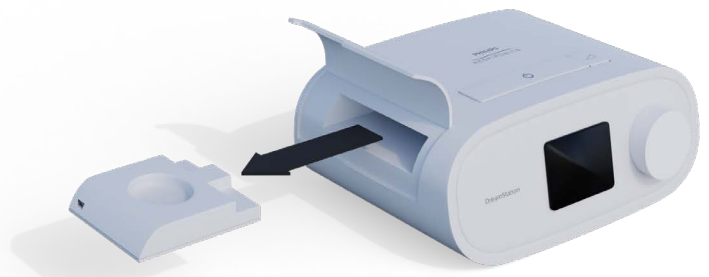


*Please adhere to your local council's laws and regulations when disposing of electronic devices.

B. Instructions for installing your DreamStation 4G cellular modem

Ensure your DreamStation device is turned off.

1. Open the modem accessory door located on the left side.

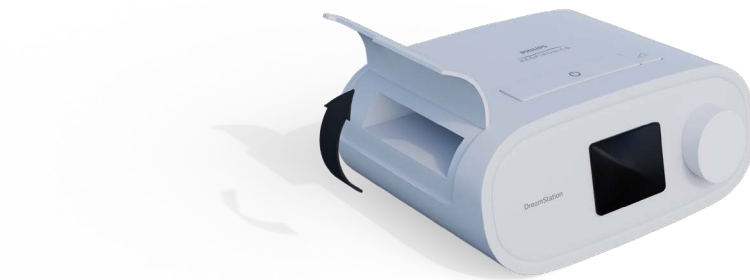

2. Remove the modem accessory door from your DreamStation device by lifting and firmly sliding the door to the left and pulling the right side of the door toward you until it pops out. This may take several attempts.



3. Slide the combined 4G cellular modem and door into the accessory slot until it clicks into place.



Note: After powering your DreamStation device, up to 30 seconds may elapse before the modem is ready to make a call.

Note: The modem will automatically call within 5 minutes of installation if the blower is off and a manual call has not been successfully completed.

ALWAYS FOLLOW THE DIRECTIONS FOR USE.

© 2025 Koninklijke Philips N.V. All rights reserved. Specifications are subject to change without notice. Trademarks are the property of Koninklijke Philips N.V. (Royal Philips) or their respective owners.

P00001126-00-00
Version: 07/2025 ANZ



Philips Sleep and Respiratory Care
65 Epping Road North Ryde, NSW, 2113
Australia

www.philips.com.au/sleep

Telephone: 1800 830 517 (toll-free)